

PAJUNK®

*MultiStim SWITCH –
Safe and Precise*



Plexus and Epidural Anaesthesia

MultiStim SWITCH

Brings the future of stimulation right to the point

With MultiStim SWITCH, PAJUNK® has set new standards in electrical nerve stimulation. Regarding safety and basic construction, the device is comparable with the MultiStim SENSOR, it is, however,

different in following four functions. This expansion of the scope of performance opens up new ways for you in nerve stimulation.



Simple changeover by CATH button

By pressing the CATH button, the stimulation can be switched between cannula and stimulating catheter, depending on which is required.

Simultaneous change of current intensity and pulse width

The new MultiStim SWITCH has an internally programmed function, which will alter the current intensity and the pulse width simultaneously. This option ensures, that the anaesthetist can approach the nerve in constant steps under continued muscular response, and can therefore concentrate his full attention completely on the puncture.



Patient resistance under control

The patient resistance is continuously displayed by the stimulator during the complete puncture. In case of an intraneural or intravascular puncture, the resistance will increase suddenly and trigger an acoustic as well as a visual signal.

The MultiStim SWITCH is therefore the first nerve stimulator which permits the anaesthetist to react immediately.



Maximum voltage and current intensity

The device emits a stimulation voltage of at most 95 V_{pp} in order to still obtain a good stimulation effect on patients with high resistance conditions. The intensity of the stimulation current for the catheter amounts to 20 mA for safety reasons, and is therefore also suitable for epidural stimulation.

All the information at a glance:

Nerve stimulator: MultiStim SWITCH

Device type:	BF	Stimulation voltage:	max. 95 V _{pp}
Battery:	9 V	Stimulation frequency:	1 Hz, 2 Hz
Current intensity:	max. 6 mA _{pp} / 20 mA _{pp}		

Product	Item no.
MultiStim SWITCH	
incl. equipment case and SWITCH cable for the connection of the cannula and the StimuLong catheter	1151-94-40
Patients cable	1151-94-07



تجهیز گستر آگرین
نماینده انحصاری PAJUNK در ایران
تلفن: ۰۲۱-۸۸۶۰۵۷۳۱-۲
agrin@agrin.ir
www.Agrin.ir

PAJUNK GmbH
Medizintechnologie
Karl-Hall-Strasse 1
D-78187 Geisingen/Germany
Phone +49 (0) 77 04/92 91-0
Fax +49 (0) 77 04/92 91-6 00
www.pajunk.com